


Wear Your Solo Anywhere You Inject or Infuse Insulin

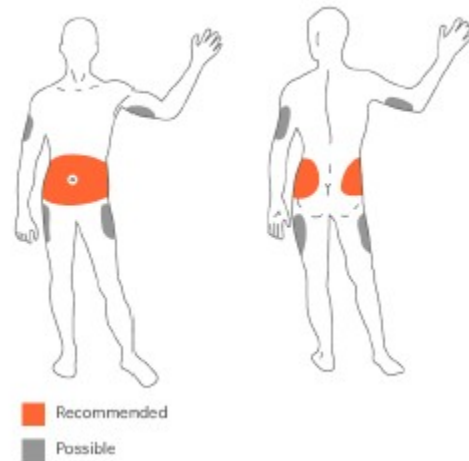

Insertion Site Tips and Cautions:

- If you are allergic to acrylic adhesives or teflon, consult with your Healthcare Provider for advice before using any insulin pump set. I
- Change your site early in the day so you can check your blood sugar 1-2 hours after insertion to be sure insulin is being delivered properly – not right before bedtime. C
- Place your MicroPump where it will be easy to press the Pump Bolus buttons if you want to use them. P
- Avoid insertion sites where belts, waistbands, or tight clothing may rub against the MicroPump – or where you might easily bump it off. A
- A new infusion site should be at least 2" (5 cm) away from the last site. A
- Do **not** place the MicroPump within 2" (5 cm) of your belly button/waist. D

- void highly sensitive or highly mobile areas. A

- o **not** apply the Cradle over scar tissue, lipohypertrophy, body piercing, moles, surgical scars, tattoos, liposuction site, injured skin, bruising, rash or over a protruding bone. D

Attachment of the Solo™ MicroPump

